Error Codes and Symbols:

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
	Low Battery—when battery symbol remains on display screen during testing.	Replace the battery as soon as possible.
	Temperature out-of-range.	Move to an area that is within the operating range for the meter: 5°C–45°C. Let the meter and test strips adjust to this temperature for twenty minutes before testing.
	The test strip has not filled enough to give an accurate result.	Remove the test strip and repeat the test using a new test strip.

CE 0088 Manufacturing site / 製造廠:
110 Oaza-Inoshiri-aza-Nishiueno,
Wakimachi, Mima, Tokushima,
JAPAN 779-3603

Ascensia Diabetes Care Holdings AG
Peter Meien-Strasse 90
4052 Basel, Switzerland

ASCENSIA
Diabetes Care

For information on patents,
and related licenses, see:
www.patents.ascensia.com

Refer to your user guide for complete instructions including troubleshooting and error information

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
	The meter is sensing a used test strip.	Remove the test strip and retest using a new test strip. Wait until you see the flashing blood drop in the display before adding the blood sample.
	Test strip not inserted correctly.	Remove the test strip and insert it properly.
E5 E6 E8 E9	E10 E11 E12 E13	Potential software or hardware issue.
		Remove the test strip and retest using a new test strip. If you continue to have problems, contact Customer Service.

Ascensia, the Ascensia Diabetes Care logo, Clinlog, Contour, Glucofacts, and the No Coding logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

WHAT YOU SEE	WHAT IT MEANS	WHAT YOU SHOULD DO
	Wrong test strip.	Remove the test strip and retest using a CONTOUR®TS test strip.
	Test result is "HI" (above 33.3 mmol/L)	Wash your hands and the test site. Repeat the test using a new test strip.
	Test result is "LO" (below 0.6 mmol/L).	If your result is still "HI" or "LO," contact your physician or health care professional immediately.

CAUTION: Glucose levels above 13.9 mmol/L or below 2.8 mmol/L may indicate a potentially serious medical condition.

Ascensia Diabetes Care
Hong Kong Limited
Unit Nos. 12-13, Level 31, Tower 1,
Millennium City 1
No. 388 Kwun Tong Road,
Kwun Tong, Kowloon,
Hong Kong
Tel: (852) 8100 6386
Fax: (852) 2320 7022

Ascensia Diabetes Care
Singapore Pte Ltd
600 North Bridge Road
#06-01 Parkview Square
Singapore 188778
Tel: +65 6202 4270
Email: diabetescare@ascensia.com

Ascensia Diabetes Care
Malaysia Sdn Bhd
Lot 2-2, Level 2, Tower B
The Troika
19 Persiaran KLOEC
50450 Kuala Lumpur
Malaysia
Tel: (60) 3 2178 6162

www.diabetes.ascensia.com

©2017 Ascensia Diabetes Care Holdings AG. All rights reserved. Rev. 06/17

Contour®
Blood Glucose Monitoring System **TS**

QUICK REFERENCE GUIDE
Read your user guide and lancing device insert for complete instructions before performing a blood glucose test.

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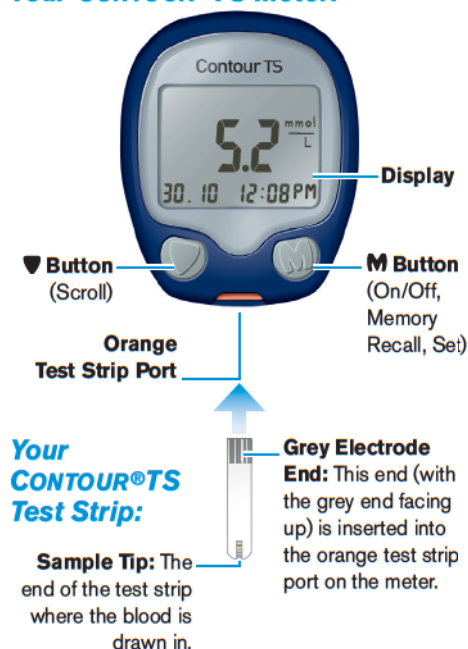
Your CONTOUR®TS Meter:

Your CONTOUR®TS Test Strip:

Sample Tip: The end of the test strip where the blood is drawn in.

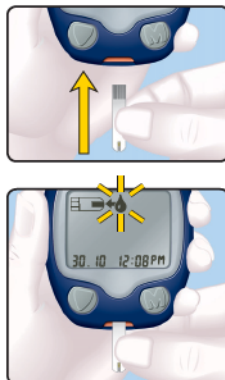
- Always wash your hands well with soap and water before and after testing, handling the meter, lancing device or test strips.
- The lancing device provided with your kit is intended for self testing by a single patient. It must not be used on more than one person due to the risk of infection.

Inserting the Test Strip:

Remove a test strip from the bottle and **firmly snap the lid closed. Never store test strips outside the bottle!**

Hold the test strip with the grey end facing up and insert it into the orange test strip port. **No coding required!**

A test strip with a flashing blood drop will appear letting you know the meter is ready to test.


Getting the Blood Drop:

Press the lancing device firmly against the puncture site and press the release button.

The lancing device graphics are for illustration purposes only.

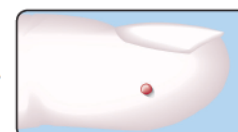
Test immediately after you have formed a blood drop. Only 0.6 µL is needed for a test.

Suggested Actual Drop Size

Testing Your Blood:

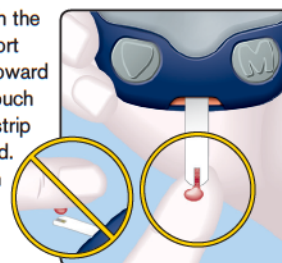Hold the meter with the orange test strip port pointing down or toward you. Immediately touch the **tip** of the test strip to the drop of blood. The blood is drawn into the test strip through the tip.

Hold the tip of the test strip in the blood drop until the meter beeps and the 5 second countdown begins.

To turn your meter off, simply remove the test strip. Dispose of the test strip as medical waste or as advised by your health care professional.

To remove the lancet from the lancing device refer to the detailed instructions in the lancing device insert.

For further assistance please contact your local Ascensia Diabetes Care agency.
www.diabetes.ascensia.com

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